

To schedule a same day test,
scan the QR code or visit
www.carolinapharmacy.com/oraltox



OralTox Rapid Drug Screening Workplace Product Solutions

ORALTOX RAPID ORAL FLUID

*OT-80404 - 4 PANEL: AMP, COC, OPI, PCP
*OT-80502 - 5 PANEL: AMP, COC, MET, OPI, THC
*OT-80504 - 5 PANEL: AMP, COC, OPI, PCP, THC
*OT-80505 - 5 PANEL: AMP, COC, MET, OPI, OXY
*OT-80508 - 5 PANEL: AMP, COC, MET, OPI, PCP
*OT-80510 - 5 PANEL: COC, MET, OPI, OXY, THC
OT-80600 - 6 PANEL: AMP, BZO, COC, MET, OPI, THC
*OT-80602 - 6 PANEL: AMP, COC, MET, OPI, PCP, THC
*OT-80605 - 6 PANEL: AMP, COC, MET, OPI, OXY, THC
*OT-80622 - 6 PANEL: COC, MET, OPI, OXY, PCP, THC
*OT-80701-IP - 7 PANEL: AMP, COC, MET, OPI, OXY, PCP, THC
*OT-80807 - 8 PANEL: AMP, COC, MET, MTD, OPI, OXY, PCP, THC
**OT-80901 - 9 PANEL: AMP, BAR, BZO, COC, MET, MTD, OPI, OXY, PCP

****OT-81053 - 10 PANEL: AMP, BAR, BZO, COC, MET, MTD, OPI, OXY, PCP, THC (\$39.00 per test)**

*Allow us to provide you with **RELIABLE, COST-EFFECTIVE,**
and **SAME DAY** drug screening results.*

Hidden Panel Preview



FOR MORE INFORMATION

Contact chirag@carolinapharmacy.com
or simply use the QR code above to schedule a drug screening.

AMP

*Amphetamine
50ng/mL

BAR

Barbiturates
50ng/mL

BUP

Buprenorphine
5ng/mL

BZO

Benzodiazepines
10ng/mL

COC

*Cocaine
20ng/mL

COT

Cotinine
30ng/mL

FYL

Fentanyl
10ng/mL

K2 (Spice)

Synthetic Marijuana
30ng/mL

KET

Ketamine
30ng/mL

MET

*Methamphetamine
50ng/mL

MTD

**Methadone
30ng/mL

OPI

*Opiates
40ng/mL

OXY

*Oxycodone
20ng/mL

PCP

*Phencyclidine
10ng/mL

THC

*Marijuana
40ng/mL

TML

Tramadol
30ng/mL


*FDA 510(k) Cleared (For in vitro diagnostic use for point-of-care)

**Employment and Insurance Use Only (workplace drug testing) and/or Forensic Use Only with proper credentials
required for law enforcement, corrections, and/or state/federal government locations.

Convenient Testing Labs open up to 7 days a week, 8am-9pm in Charlotte, Rock Hill, and Lancaster

Components and Test Preparation


The donor must avoid placing anything in their mouth for at least 10 minutes prior to the collection. This includes food, drink, gum, and tobacco products.

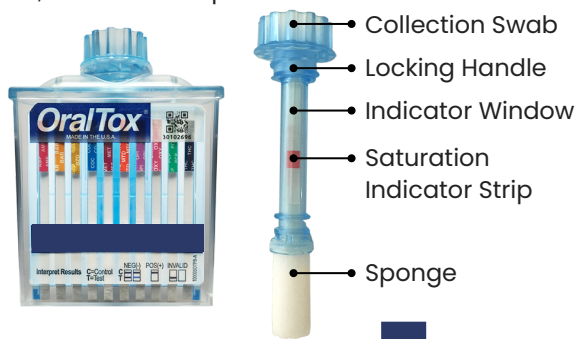

Specimen Collection

Have the donor sweep inside of mouth (cheeks, gums, and tongue) and then hold the swab in mouth until the color appears in the indicator window. It is important not to bite, suck, or chew on the sponge. If after 7 minutes the saturation indicator has not turned color, discard the device and repeat the test.

Donor may remove swab from mouth when color appears in the indicator window.

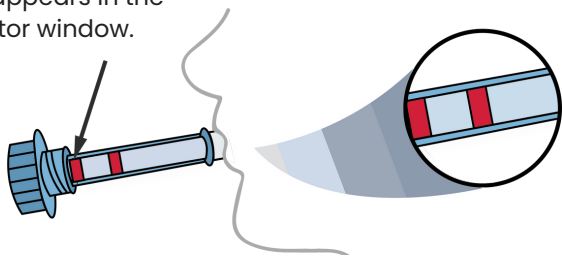


Test Activation

With the test device positioned upright on a **FLAT SURFACE** (test device remains in this position until results interpreted), the donor removes the collection swab from their mouth and hands it to the administrator. The administrator inserts the collection swab into the device sponge first, pushing and twisting down until the threaded handle locks in place.


Test Interpretation

3

Negative results can be read as soon as the test and control lines appear on any test strip (often within 2 minutes).

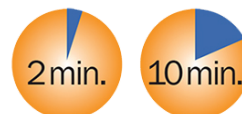

Presumptive positive results are to be read at 10 minutes. Interpret test as negative, presumptive positive, or invalid. Do not read test results after 20 minutes. Follow your organization's established protocol for presumptive positive results. After the results have been interpreted, the OralTox device may be thrown in a regular receptacle.

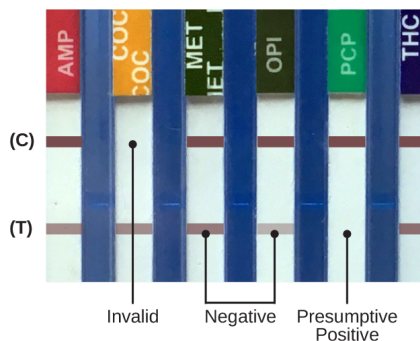

The control line is the uppermost line appearing in each test area. Before reading the test result lines, verify that all control lines have formed. Results from any test without a top control line is **INVALID** and the test must be discarded.

Negative

A **NEGATIVE** result is indicated by two colored bands appearing on the membrane. One band appears in the control region (C), and another band appears in the test region (T). The intensity of the test lines may vary. Any line, without regard to intensity, color, or size, is a line and indicated a negative result for that drug.

Presumptive Positive

A **PRESUMPTIVE POSITIVE** result is indicated when only one colored band appears in the control region (C), and no apparent colored band appearing in the test region (T).

AMP
*Amphetamine 50ng/mL
BAR
Barbiturates 50ng/mL
BUP
Buprenorphine 5ng/mL
BZO
Benzodiazepines 10ng/mL
COC
*Cocaine 20ng/mL
COT
Cotinine 30ng/mL
FYL
Fentanyl 10ng/mL
K2 (Spice)
Synthetic Marijuana 30ng/mL
KET
Ketamine 30ng/mL
MET
*Methamphetamine 50ng/mL
MTD
**Methadone 30ng/mL
OPI
*Opiates 40ng/mL
OXY
*Oxycodone 20ng/mL
PCP
*Phencyclidine 10ng/mL
THC
*Marijuana 40ng/mL
TML
Tramadol 30ng/mL

*FDA 510(k) Cleared